




Childrens Respiratory Monitor


Guide for Healthcare Professionals

Testing Subject

1 Insert Mouthpiece and 1 Switch On

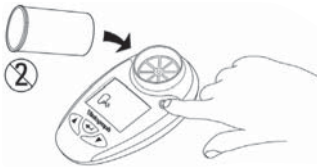

2 Blow 3 times

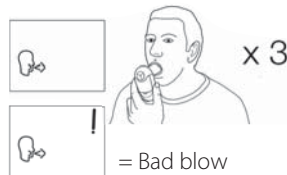

3 Best blow



PEF +/- 5%; FEV₁ +/- 3%

Set PEF Personal Best

1 1 Switch On



2 ▼ + ▲ hold for 3 seconds



3 ▲ Increase PEF



4 ← Set PEF best



Press ← ← to exit or

Set FEV₁ Best

5 ▲ Increase FEV₁



6 ← Set FEV₁ best



Note 1:

Zones set at 80% and 50%

Note 2:

Follow the same method for FEV₅ and FEV₇₅ Best

EN ISO 23747:2007

Specifies accuracy: ± 10% PEF

Resistance to flow: < 0.35 kPa/L/s

Vitalograph Childrens Respiratory Monitor

Accuracy: ± 5% PEF

± 3% FEV₁, -5/-75

Resistance to flow: < 0.15kPa/L/s

Review History

1 1 Switch On



2 ← Hold for 3 seconds



3 ▲ or ▼ Review History



4 ← Return



Delete History

1 ① Switch on



2 ▼ + ← Hold for 10 seconds



Cleaning & disinfecting:

The outer surfaces should be cleaned every week, more often if necessary. The use of a disposable disinfecting wipe is recommended, with special attention to the mouthpiece area.

Modify Green Zone

1 ① Switch On



2 ← + ▲ Hold for 3 seconds



3 ▲ or ▼ Upper Boundary



4 ← Set Upper Boundary



Modify Other Zones

5 ▲ or ▼ Lower Boundary



6 ← Set Lower Boundary



7 ▲ or ▼ Mid Boundary



8 ← Set Mid Boundary



Note: 3-Zones now set.
Press ← ← to exit or for 4-Zones...

Default Zone Settings

1 ① Switch On



2 ▲ + ▼ + ← Hold for 10 seconds



3 ① Switch Off



Vitalograph®

Your respiratory partner

UK Sales
Vitalograph Ltd.
Maids Moreton Buckingham
MK18 1SW England
Tel: 01280 827110
Fax: 01280 823302
e-mail: sales@vitalograph.co.uk
www.vitalograph.co.uk

Export Sales
Vitalograph Ltd.
Maids Moreton Buckingham
MK18 1SW England
Tel: +44 1280 827120
Fax: +44 1280 823302
e-mail: sales@vitalograph.co.uk
www.vitalograph.eu

Vitalograph GmbH
Rellinger Straße 64a
20257 Hamburg
Germany
Tel: (040) 547391-0
Fax: (040) 547391-40
e-mail: info@vitalograph.de
www.vitalograph.de

Vitalograph Inc.
13310 West 99th Street
Lenexa
Kansas 66215 USA
Tel: (913) 888 4221
Fax: (913) 888 4259
e-mail: vitcs@vitalograph.com
www.vitalograph.com

Vitalograph (Ireland) Ltd.
Gort Road Business Park
Ennis
Co Clare Ireland
Tel: (065) 6864100
Fax: (065) 6829289
e-mail: sales@vitalograph.ie
www.vitalograph.ie

www.vitalograph.co.uk